Read this instruction sheet thoroughly before use!

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419-24032
0124D6

Intended use

The Wako LDL-C Calibrator is designed to be used with the Wako LDL-C L-Type test for the determination of low density lipoprotein cholesterol (LDL-C) in serum.

Reagents and ingredients

LDL-C Calibrator: 4 bottles x for 1 mL.
Contains human serum and stabilizer.

Reagent preparation

Accurately add 1 mL of distilled or deionized water to dissolve the contents of each bottle of the Calibrator. The reconstituted Calibrator is stable for 7 days at 2–10 °C.

Assigned values and their determination

Assigned values: See bottle labels.
Determination: Based on the US CDC LDL-C Reference method.

Storage condition

Store at 2–10 °C.

Warnings and precautions

- For *in vitro* diagnostic use.
- Only for professional use.
- Not to be used internally in humans or animals.
- Do not use the calibrator after the date of expiry indicated on the packaging.
- Do not use this calibrator for any other purpose than described herein.
- Human serum material was used in the manufacture of this calibrator. The raw human serum used was tested and found negative for HBsAg, anti-HIV antibody and anti-HCV antibody. Because no test can offer complete assurance that products derived from human blood will not transmit infectious agents, it is recommended that the calibrator should be handled with the same biohazard precautions used for patient specimens.
- When discarding the product, do so in accordance with local and national regulation.
- This kit contains components classified as follows according to the Regulation (EC) No 1272/2008.

Labelling according to Regulation (EC) No 1272/2008

- The product is classified and labelled according to the CLP regulation.
- The product does not contain a SVHC according to REACH, Article 57 >0,1 %

Hazard-determining components of labelling

- Globulins, γ-
- Mixture of 5-Chloro-2-methyl-2H-isothiazol-3-one and 2-methyl-2H-isothiazol-3-one

Hazard pictograms

Warning

Hazard statements

- Causes serious eye irritation.
- May cause an allergic skin reaction.
- May cause respiratory irritation.
- Harmful to aquatic life with long lasting effects.

Precautionary statements

- Avoid breathing dust/fume/gas/mist/vapours/spray.
- Wash thoroughly after handling.
- Avoid release to the environment.
- Wear protective gloves / eye protection.
- IF exposed or concerned: Call a POISON CENTER/doctor.

Ordering information

Code No.	Product	Package
419-24032	LDL-C Calibrator	CAL: 4 x for 1 mL

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